vibbel

0009361

# Khona Press 3 in 1

Pressotherapy + Thermotherapy + Electrostimulation / Presoterapia + Termoterapia + Electroestimulación  
/ Pressothérapie + Thermothérapie + Électrostimulation / Pressoterapia + Termoterapia +  
Elettrostimolazione / Pressotherapie + Thermotherapie + Elektrostimulation

EN	2
ES	8
FR	14
IT	20
DE	26

DDUUEETT EUROPE would like to thank you for choosing Vibbel products. We hope that this instruction manual will help you get the most out of this product.

If you have any doubts regarding its use and/or operation, please contact your distributor.

## BEFORE USE

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

## CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation of DDUUEETT EUROPE SL.
- DDUUEETT EUROPE SL shall not be held liable for any material or personal damage due to the misuse of the product.

## CONTRAINDICATIONS

- This product should not be used on:
  - People who suffer from deep vein thrombosis.
  - People with pacemakers or acute heart failure.
  - People with high blood pressure.
  - Pregnant women.
  - People with wounds or injuries in the area to be treated.
  - People with pain, irritation, pins and needles, cramp or numbness in the area to be treated.
  - People with metal implants.
- Not recommended for cancer patients or people with an overactive thyroid.
- The following should check with their doctor:
  - People with skin disorders.
  - People undergoing medical treatment or a diagnostic process.

## PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement weather.
- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

## SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device.
- Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

## CONTENTS

- Main body x 1 unit
- Power cable x 1 unit
- Suit: 2 boots, 2 legs, 2 arms and waist. With thermotherapy connectors
- Mat. With pressotherapy and thermotherapy connectors
- 10 round electrodes: 6 large / 4 small
- 10 elastic bands of different sizes
- Set of cables for electrostimulation
- Fuse x 2 units
- Instruction manual x 1 unit

## TECHNICAL SPECIFICATIONS

Product	Pressotherapy + Thermotherapy + Electrostimulation
Reference	0009361
Voltage	AC 200-240 V ~
Input frequency	50 Hz
Power	350 W
Dimensions	44 x 31 x 15 cm

## COMPONENTS

- Main body

- Suit, Pressotherapy and thermotherapy function


Section ID	Section of the Suit	Thermotherapy Function Connector Number
R-0-1	Right boot	2
L-0-1	Left boot	1
R-2-3-4-5	Right leg	2
L-2-3-4-5	Left leg	1
R-8-9	Right arm	4
L-8-9	Left arm	3
R-6-7 / L-6-7	Waist	5

- Mat. With pressotherapy and thermotherapy connectors

- 1 Air pressure tube connector
- 2 Thermotherapy connector

- Electrostimulation Accessories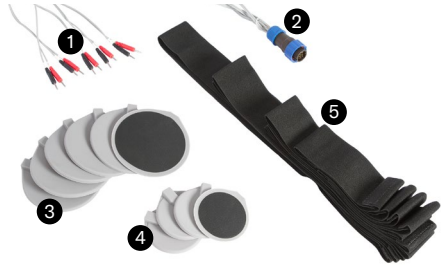

- 1 Electrostimulation cables (numbered in pairs, red and black from 1 to 5)
- 2 Electrostimulation connector
- 3 Large round electrode pads
- 4 Small round electrode pads
- 5 Elastic bands of different sizes

## CONTROL PANEL

- 1 On / Off button
- 2 Display screen and parameter control



## REAR PANEL


- 1 Air pressure tube connector
- 2 Electrostimulation connector
- 3 Thermotherapy connector
- 4 Electrical power supply connector
- 5 Fuse

## CONNECTION AND ASSEMBLY INSTRUCTIONS

- Connect the power cable to the rear panel of the main body.
- Connect the piece coming from the mat (the one with a mobile revolving cap on one end) to the air pressure tube connector located on the rear panel of the main body. Insert the piece and then twist the cap until it feels tight enough.
- Insert the thermotherapy connector into the back panel of the main body.
- Insert the electrostimulation connector into the back panel of the main body.
- Connect the pressure tubes housed within the mat to the various sections of the suit. A number will indicate where each tube should be connected.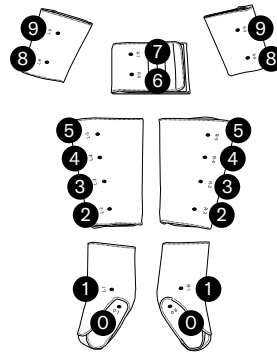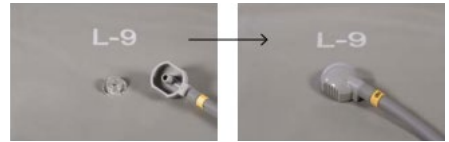

- Connect the thermotherapy cables housed in the blanket with the corresponding cables found on the different sections of the suit. In the previous section of this manual "Suit. Pressotherapy and thermotherapy function" it indicates which section of the suit you should connect to the cables of the thermotherapy function. The sections can be identified with the numbers 1 to 5.

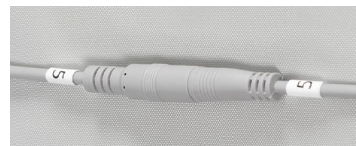- For the use of the electrostimulation function:
  - The cables are numbered in pairs. Each pair is formed of a red and a black cable with the same number, from 1 to 5.
  - Connect each pair of cables to a pair of electrodes.
  - Place each pair of electrodes on the area of the body that needs treatment, using the correct sized elastic band, so that they are secure.
- Always apply conductive gel between the electrode and the skin.

## POWER AND CONFIGURATION

- Press the on/off button to start the device. When it is on, a beep will sound and the welcome screen will be displayed.
  - Press the screen to move to the next display screen and the parameter control.

## **RECYCLING**

If you need to dispose of the product or any of its parts or accessories, please check and respect the laws and regulations applicable to waste management and environmental protection.

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